

## Structure of the menu

The menu tree is set together below. Individual items are described in detail in the chapter 7.

Symbols applied in the graph:  
 Mode of availability of a function:  
 < - out of the session,  
 ² - during the session,  
 ; - during the pause.  
 If none of the foregoing icons appears, a function is available in every mode.

Context, for which the value of a parameter is in force:  
 ☉ - for a given user,  
 • - for a given protocol.  
 If none of these icons appears, the parameter is in force irrespective of a context (the same value for all users and protocols).

Other symbols:  
 ! - command,  
 ü - default value of a parameter.  
 Context dependant in case of no default value signed for a given parameter.

